

# Black Jack

## Switch

### User Manual



### Description

---

The Black Jack Switch was created as a simple and effective means to power and switch on/off any device that is powered through a regular audio cable (3.5mm). There are four output jacks. There is also an auxiliary switch jack. Use this to connect any preferred switch to run the unit. *(The auxiliary switch jack accepts a 3.5mm audio cable)*

#### Parts List :

- 1- Switch Box
- 1- Power Cable

### Installation and Use

---

- Step 1 Remove the wheelchair's battery case cover. Some battery boxes have pre-cut wire channels which will allow the power cable's quick disconnect wires to neatly come out of the battery box. Other battery boxes may require the user to have a new hole drilled into the battery case. **If you have to drill a new hole in your wheelchair's battery case, remove the battery from the case before drilling!**
- Step 2 Feed the wires through the new hole (or pre-cut wire channel) on your wheelchair's battery box.
- Step 3 Identify the Battery Cable's positive and negative ring terminals:  
**Positive = Red wire, Ground = Brown wire.**
- Step 4 Connect the **Ground** ring terminal to the battery system's ground (0 volts). Connect the **Positive** ring terminal to the same battery's positive (+12 volts). This will span across only one battery.
- Step 5 **STOP:** Double check your wiring to ensure that you have observed proper battery polarity. Check for proper power at output of power cable.
- Step 6 Replace your wheelchair's battery box cover. Be sure not to inadvertently crimp or cut the power cable when replacing the battery case. **The battery cover should go back on as easily as it came off.**
- Step 7 Tidy up all excess slack and loose wire. Be Neat!

## Technical Specifications

---

Current: 3 amps max  
Output Jacks: cable tip- Positive (+)  
cable base- Ground (-)  
Auxiliary Switch: normally open switch; connects cable tip to base

## Service

---

There are no serviceable parts inside the Black Jack Switch. A fuse is located inline with the power cable near the ring terminals. Please contact Richardson Products Inc. about any problems.

### Fuse Replacement :

10 amp, 250 volt fast acting glass fuse (20mm x 5mm)  
www.Mouser.com part#: 576-0217010.hxp

## Related Products

---

product #	description
06079W	IllumiRay- White
06079R	IllumiRay- Red
060713	Ni-Cd Battery Pack

## Warranty and Disclaimer

---

Richardson Products Inc. warrants that the Black Jack Switch shall be free from defects of workmanship and materials and will perform in accordance with the product specifications for a period of one (1) year from the date of sale by Richardson Products Inc. to the dealer. If the product fails to perform in accordance with the product specifications, Richardson Products Inc. will repair or replace, at its discretion, the defective material or part. This warranty does not cover damage caused by accident, misuse, abuse, alteration and other defects not related to material or workmanship.

Richardson Products Inc disclaims all liability for economic loss, loss of profits, overhead or consequential damages which may be claimed to arise from any sale or use of this product.



9408 Gulfstream Road Frankfort, Illinois 60423

Website: [www.RichardsonProducts.com](http://www.RichardsonProducts.com)  
Email Address: [rpi@RichardsonProducts.com](mailto:rpi@RichardsonProducts.com)  
Phone: 815.464.3575 Fax: 815.464.3576

# Black Jack

## Switch

### User Manual



### Description

---

The Black Jack Switch was created as a simple and effective means to power and switch on/off any device that is powered through a regular audio cable (3.5mm). There are four output jacks. There is also an auxiliary switch jack. Use this to connect any preferred switch to run the unit. *(The auxiliary switch jack accepts a 3.5mm audio cable)*

#### Parts List :

- 1- Switch Box
- 1- Power Cable

### Installation and Use

---

- Step 1 Remove the wheelchair's battery case cover. Some battery boxes have pre-cut wire channels which will allow the power cable's quick disconnect wires to neatly come out of the battery box. Other battery boxes may require the user to have a new hole drilled into the battery case. **If you have to drill a new hole in your wheelchair's battery case, remove the battery from the case before drilling!**
- Step 2 Feed the wires through the new hole (or pre-cut wire channel) on your wheelchair's battery box.
- Step 3 Identify the Battery Cable's positive and negative ring terminals:  
**Positive = Red wire, Ground = Brown wire.**
- Step 4 Connect the **Ground** ring terminal to the battery system's ground (0 volts). Connect the **Positive** ring terminal to the same battery's positive (+12 volts). This will span across only one battery.
- Step 5 **STOP:** Double check your wiring to ensure that you have observed proper battery polarity. Check for proper power at output of power cable.
- Step 6 Replace your wheelchair's battery box cover. Be sure not to inadvertently crimp or cut the power cable when replacing the battery case. **The battery cover should go back on as easily as it came off.**
- Step 7 Tidy up all excess slack and loose wire. Be Neat!

## Technical Specifications

---

Current: 3 amps max  
Output Jacks: cable tip- Positive (+)  
cable base- Ground (-)  
Auxiliary Switch: normally open switch; connects cable tip to base

## Service

---

There are no serviceable parts inside the Black Jack Switch. A fuse is located inline with the power cable near the ring terminals. Please contact Richardson Products Inc. about any problems.

### Fuse Replacement :

10 amp, 250 volt fast acting glass fuse (20mm x 5mm)  
www.Mouser.com part#: 576-0217010.hxp

## Related Products

---

product #	description
06079W	IllumiRay- White
06079R	IllumiRay- Red
060713	Ni-Cd Battery Pack

## Warranty and Disclaimer

---

Richardson Products Inc. warrants that the Black Jack Switch shall be free from defects of workmanship and materials and will perform in accordance with the product specifications for a period of one (1) year from the date of sale by Richardson Products Inc. to the dealer. If the product fails to perform in accordance with the product specifications, Richardson Products Inc. will repair or replace, at its discretion, the defective material or part. This warranty does not cover damage caused by accident, misuse, abuse, alteration and other defects not related to material or workmanship.

Richardson Products Inc disclaims all liability for economic loss, loss of profits, overhead or consequential damages which may be claimed to arise from any sale or use of this product.



9408 Gulfstream Road Frankfort, Illinois 60423

Website: [www.RichardsonProducts.com](http://www.RichardsonProducts.com)  
Email Address: [rpi@RichardsonProducts.com](mailto:rpi@RichardsonProducts.com)  
Phone: 815.464.3575 Fax: 815.464.3576