


Meal Buddy Robotic Assist Feeder

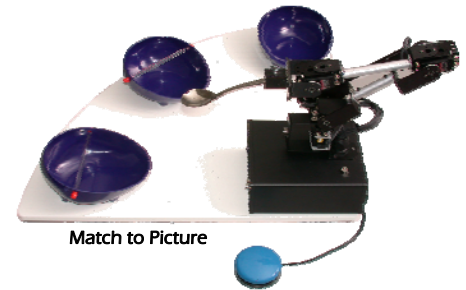

Set up

1. Set bowls and Robotic Arm onto the magnetic anchor points of the Base Board.
2. Plug power cord into wall outlet and into the back panel of the Meal Buddy.
3. Plug **Blue Button** cord into back panel of the Meal Buddy
4. Clamp the spoon completely into Robotic Arm's sleeve.
5. Turn Meal Buddy on by using switch on front panel.
6. Once you see the Mode Light turn **Green**, Meal Buddy is ready.



Teaching Mouth Position

1. Hold L2 Button and press R1 Button to bring the Robotic Arm to current feeding position.
2. Have the Patient pull up to the spoon.
3. Hold L2 Button and move Control Stick until spoon is just outside the Patient's mouth.
4. Hold L2 Button and press R2 Button to Teach Mouth Position.

Left Control Stick		Right Control Stick	
	Elbow Up/Down		Wrist Up/Down
	Shoulder Left/Right		Rotate Base Left/Right
	Invert Joystick		Invert Joystick

Eating Program

1. Press the **Blue Button** to start feeding cycle.
2. Patient can now eat.
3. Pause the cycle by tapping the **Blue Button** while Meal Buddy is moving.
4. To Stop the cycle, press the **Blue Button** again after the spoon exits the patients mouth and panel light is **Green**.

Options

Press the Start Button to turn on the Hand Controller.

Bowl Select Mode- Enables the patient to choose which bowl to eat from with every scoop.


- Choose a bowl by pressing the **Blue Button** when the Meal Buddy beeps the number of times corresponding to the bowl number you wish to eat from. (Example: 2 Beeps – Bowl number 2) If you do not select any bowls, you can start over by pressing the **Blue Button** again.

Re-Scoop- During the feeding cycle, patient can choose to re-scoop by pressing the **Blue Button** when the light is **Green**.

Dump- During the feeding cycle, the patient can dump the spoonful by pressing the **Blue Button** when the light is **Red**.

Bowl Number Selection- Increase/Decrease the number of bowls being used by using the Arrow Buttons.


Basic Controls

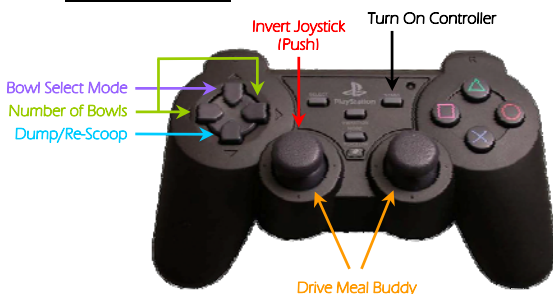

Basic Controls			
SELECT	Same as Blue Button	START	Turn on Controller
	Dump/Re-Scoop		Home Position
	Bowl Select Mode On/Off		Random On/Off
	Bowl Number Select 3 - 2 - 1		Wipe On/Off
	Bowl Number Select 1 - 2 - 3		AutoCycle On/Off
L1	N/A	R1	N/A
L2	Alternate Controls (Hold Down)	R2	N/A

Alternate Controls	
	Food Level Select Less/More
	Speed Level Select Slow/Fast
R1	Remember Mouth Position
R2	Teach Mouth Position
	Drive Meal Buddy
	Invert Joysticks
L1	Factory Reset

Alternate Controls

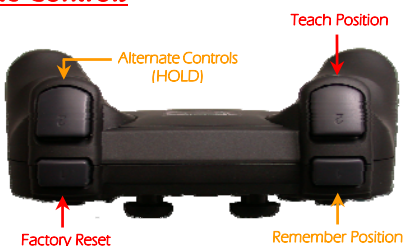

For a Complete Guide, Please See the User Manual